INSTRUCTION MANUAL/MANUAL DE INSTRUCCIONES/MANUEL D'UTILISATION/ MANUALE D'ISTRUZIONI/ANWEISUNGEN: Rollator Emerita

Última revisión: 22 de Marzo 2021
Last Revision Date: March 22th 2021

MOBICLINIC
Spanish Leading Company Since 1985

ENGLISH

Please read carefully this direction to know your new rollator well before use. Emerita model has a weight capacity of 100kg.

The followings are steps of using rollators:

1. Place both feet side-by-side in the middle of the rollator frame.
2. Grasp the hand grips on either side of the rollator.
3. Push the rollator forward to a comfortable distance.
4. Step forward toward the inside of the rollator frame with your weaker leg.
5. Step forward with your stronger leg while supporting yourself with the hand grips.

Tips and Warnings

Keep a part of your body, like your toes, within the rollator frame at all times. When standing from a sitting position, push up from the seat instead of pulling up on the rollator handles. When turning, take small steps and keep your hips parallel to the front bar of the rollator. Avoid twisting your body away from the rollator. Don't sit on the rollator seat and don't use this device as a temporary wheelchair. The rollator can tip over causing injury. Ensure all wheels of the rollator are on level ground before walking. Don't hunch forward over the rollator. Don't use the rollator on an escalator or stairs.

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JIANLIAN HOMECARE PRODUCTS CO., LTD
LOTUS NL B.V.
Imported by/Importado por: Grupo R. Queraltó S.A.